

**General** These commands may be used in the following modules:  
 Calcium, 3D Viewer, TDA, Vessel Analysis, SAT, Colon Unfold, Patient List, and 2D Viewer.

Change the orientation:

**A** **P** **R** **L** **F** **H**

= Rotate or Page

= Zoom

= Pan

Window/Level Presets:

**1** **2** **3** **4** **5** **6**

To modify a window/level preset, adjust the W/L to the desired setting, and then right click on the preset button you wish to program in the W/L toolbar on-screen.

= Window/Level

= Stack or Page

## Fly-Through

### OPERATIONS

**C** = Capture the main image.

**Delete** = Delete the selected measurement.

**Space Bar** = Stop Flying

**Tab** Hold + Click + Drag to add an arrow.

**V** = Open Unfold

**Backspace** = U-Turn

**F7** / **F8** = Go to the previous/next unviewed area.  
 Works only when "List Unviewed" is opened.

**MEASUREMENT** Toggle the keys to return to the previous mouse mode.

**M** = Toggle the arrow mode.

**D** = Toggle the distance measurement mode.

**E** = Toggle the ellipse measurement mode.



## Quick Reference Card

Version 3.5.0

For technical support, please call 1-800-746-1849 or email: support@terarecon.com.

## 3D Viewer

### OPERATIONS

**+** / **-** = Change the zoom feature.

**Ctrl** + **F** = Toggle the rendering mode to draft/fine, draft only, and fine only mode.

**C** = Capture **Ctrl** + **C** = Capture All

**Ctrl** + **Shift** + **C** = Multiple Auto-Capture

**Ctrl** + **Z** / **Z** = Quick Undo

**Alt** + = Set the center of rotation.

**→** = Rotate the main view to the right by 15 degrees.

**←** = Rotate the main view to the left by 15 degrees.

**↑** = Rotate the main view up by 15 degrees.

**↓** = Rotate the main view down by 15 degrees.

### BATCH 3D

**Ctrl** + **A** = Add a new frame to the batch.

### ARROW, LABEL, TEXT

**M** = Toggle the mouse arrow mode. **N** = New Finding

**T** = Direct Input Annotation (English text only)

**Y** = History Annotation (English text only)

**Tab** + = New Arrow

### MEASUREMENTS

Toggle the keys to return to the previous mouse mode.

**D** = Distance Measurement

**E** = Ellipse **I** = Profile

**V** = Quick Stenosis **<** = Angle

**Ctrl** + **E** = Body Fat Analysis

**Ctrl** + **D** = Distance

**Delete** = Delete the selected measurement, annotation or arrow.

## Vessel Analysis

### OPERATIONS

**C** = Capture the large view. **Delete** = Delete the selected measurement.

### MEASUREMENT *Toggle the keys to return to the previous mouse mode.*

**D** = Distance **<** = Angle

## Calcium

**↑** / **↓** = Increase or decrease the amount of images.

**F4** = Show the coronary chart image.

**F5** = Show simple color mapping.

**F7** = Select the current image for a report.

**F8** = List current selected images with an option to clear.

**R** = Reset the zoom feature.

**M** = Measure the distance.

**L** = Change the mouse mode to select lesion.

**W** = Change the mouse mode to window/level.

**S** = Change the mouse mode to page slices.

**I** = Toggle on/off the isotropic interpolation mode for slice(s).

**V** = Toggle on/off the lesion selection in the visible area only.

**O** = Toggle on/off the lesion overlay.

**C** = Toggle on/off the calcium overlay.

**H** = Display an HU map.

**P** = Setup the application parameters.

**Alt** + **F4** = Save and exit.

**Esc** = Exit without saving.

## Colon Unfold

**C** = Capture the large view.

**Delete** = Delete the selected measurement.

## SAT

**C** = Capture the main image.

**Delete** = Delete the selected measurement.

**Shift** Hold + Click to make a new finding.

**M** = Toggle the mouse to select a new finding mode.

## TVA

**→** / **↓** = Move to the next phase. **←** / **↑** = Move to the previous phase. **Ctrl** + **C** = Capture All

**M** = Toggle the mouse arrow mode.

**Tab** Hold + Click + Drag to add an arrow.

**Tab** Hold + Click to make a new finding.

**D** = Toggle the mouse to create a distance measurement mode. *Available only when a finding is selected.*

**F7** / **F8** = Select the previous/next finding.

**F9** = Toggle the mask overlay.

## TDA

**↑** / **↓** = Increase or decrease the amount of images.

**F4** = Show the option dialog.

**F5** = Change to the original image.

**F6** = Change to the MIP image.

**F7** = Change to the CBF image.

## 2D Viewer

**↑** / **↓** = Increase or decrease the amount of images.

**Delete** = Delete the selected measurement.

**Ctrl** +  Click = Select multiple measurements.

**F8** = Change to the CBV image.

**F9** = Change to the MTT image.

**F11** = Toggle on/off the annotation.

**F12** = Toggle on/off the overlay.

## Patient List

**Ctrl** + **+** = Preview the window.

**Shift** +  Double-Click = Open the selected series with the 2D filter.

**Alt** +  Double-Click = Open Window Explorer™ with the selected series.